Plot No.19 Sector 6 I.M.T. Manesar
Gurgaon Haryana 122 050 India
T: +91 124 2291406/7/8 F: +91 124 2291206
Email: info@adityadispomed.com
www.adityadispomed.com

aditya
DiSPomed
PRODUCTS
PRIVATE
LTD

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DOCUMENT IFU-01 ED01 250808

USAGE INSTRUCTIONS FOR SURGICAL BLADES

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Surgical Blades are packed in Peel Apart foil pouches. Each blade is individually packed in a foil pack to prevent the edge from getting damaged in contact with another blade. Carbon steel blades have a protective VCI liner.

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It is recommended that the foil pouch is peeled apart half in a sterile area allowing the blade to be exposed. There is no need to drop the blade from the pack. The foil pouch has an indication with marker clearly showing in which direction the foil pouch has to be peeled apart.

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The blade can be either directly fitted on to the metal handle by sliding the handle in the slot made on the blade before removing the blade from the pouch. Alternatively the blade can be removed from the pouch with a force and fitted on the handle as shown in the attached sketch.

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Proper care must be taken that the pouch is not opened in an unsterile area otherwise the blade which has already been sterilized by Gamma radiation to a minimum dosage of 2.5Mrad will become unsterile. Proper procedures must be used as applicable for handling any sterile product.

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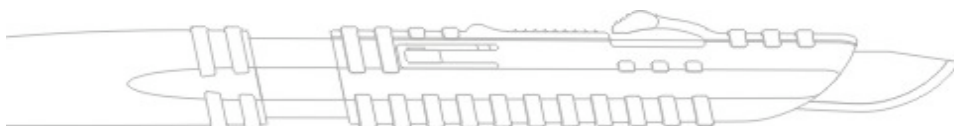
Disposal: Proper care must be taken during disposal to avoid any contact or injury due to the sharp nature of the object. Suggest dispose in an approved sharps container.

Page 1 of 3

STERILE R

CE
0 4 3 4

manufacturers and distributors of **Kiáto**

Plot No.19 Sector 6 I.M.T. Manesar
Gurgaon Haryana 122 050 India
T: +91 124 2291406/7/8 F: +91 124 2291206
Email: info@adityadispomed.com
www.adityadispomed.com

aditya
DiSPomed
RODUCTS
PRIVATE
LTD

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Surgical blades are sharp. Handle with care. Do not touch the cutting edge.

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Reference/Model Nos.: Sizes 9, 10, 10A, 11, 11P, 12, 12B, 12D, 13, 14, 15, 15A, 15B, 15C, 15T, 16, 17, 18, 19, 20, 21, 22, 22A, 23, 24, 24D, 25, 25A, 27, 34, 36, 36D, 60

REFERENZE/MODELLO No: MISURE 9,10.....

RECOMMENDED PROCEDURES FOR ATTACHMENT AND REMOVAL OF SURGICAL BLADES

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To attach blade

PER FISSARE LA LAMA

Grip blade with forceps, or similar, avoiding contact with cutting edge.

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Hold handle in left hand with fitting uppermost (fig 1).

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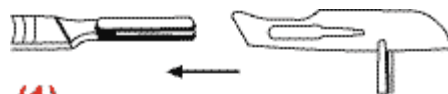
Place blade partway over handle fitting and engage slots (fig 2).

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
Slide blade until it clicks into position. (fig 3).

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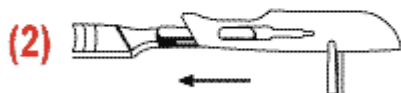STERILE R

CE
0 4 3 4

manufacturers and distributors of **KiaTo**

Plot No.19 Sector 6 I.M.T. Manesar
Gurgaon Haryana 122 050 India
T: +91 124 2291406/7/8 F: +91 124 2291206
Email: info@adityadispomed.com
www.adityadispomed.com

aditya
DiSPomed
RODUCTS
PRIVATE
LTD

To remove a blade

Grip blade with forceps, or similar.

Lift heel of blade at point 'A' with tip of index finger, avoiding contact with cutting edge, and slide away from handle carefully (fig 4).

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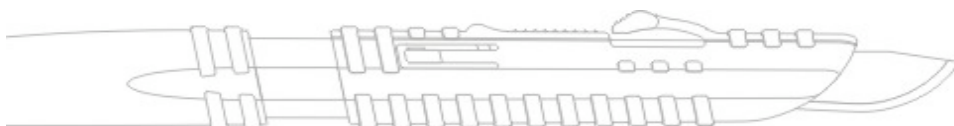
STERILE R

CE
0 4 3 4

manufacturers and distributors of **Kiaito**